


Quick Guide HeartTrends Client App Installation

1. Technical Specifications (before installation):

- Windows 7 or 8
- Microsoft [.NET Framework](#)
- Acrobat Reader
- Your Billing ID & Name
(Generated by Lev El)

2. Check connection to Lev El Server:

Check that your computer can ping Lev El server by clicking on the link:
<https://upload.levhm.com:443/server.asmx>
 the following screen will appear:

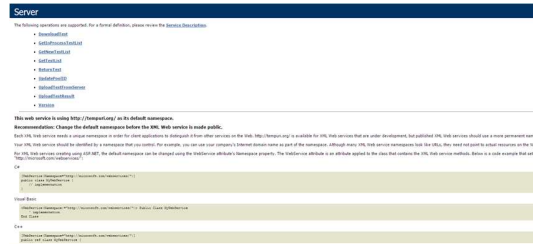


3. Install the HeartTrends Software:

The installation file will be provided by Lev El. Select language and press **Install** → **Next** → **Finish**




4. Launch the HeartTrends Software:

After the installation a Launch window will appear. Press **Launch**

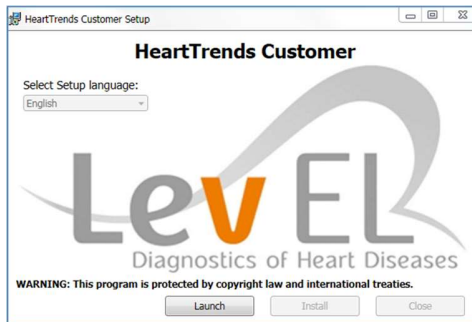

5. Insert Billing details, choose language and define your Admin password*

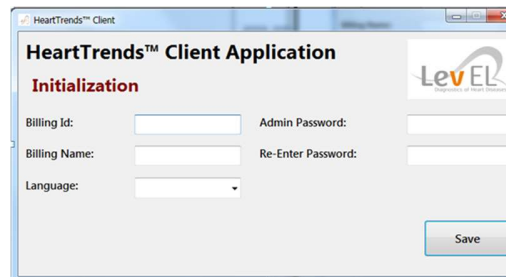

*Lev El **does not** have access to your password, therefore, maintaining the password is the responsibility of the user.

6. Press **Done** on the Holter Configuration screen, and the software will appear:

