


Quick Guide – HeartTrends Client App with Spacelabs Holter

1. Login with your user and password (the main user is **Admin**)

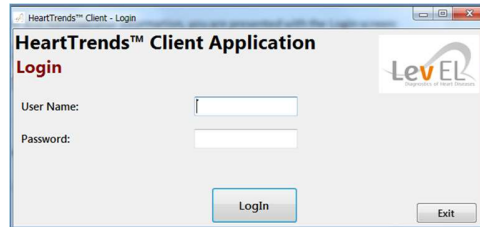

2. Make sure the CF card is connected to the computer (via card reader)




3. **First tab (New Patient)** – fill in the patient details, select holter device and press **Store**



4. Press **Yes** to condition the card (might take a few minutes), take it out of the card reader and place it inside the holter



5. Connect the electrodes to the patient according to the following drawing:

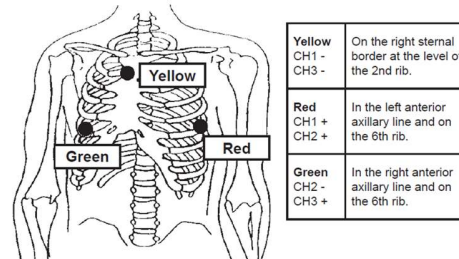

- 6.1 Connect the holter (will turn on automatically), Then press the following*:

Start... → Confirm (ECG will be shown – the holter is **NOT** recording yet!) → **START NOW**

- 6.2 The holter is now recording – during the recording a large clock appears.

- 6.3 Keep recording for minimum **60 minutes**.

*Before initial use – through holter's settings, set the holter's time and date, and **turn Speech Recording Off**.

- 7.1 Take the card out of the holter and insert it back to the card reader.
7.2 **Second tab (Download Data)** - select holter type and press **Load** → a .htd File will be created and saved on your local computer → press **OK** and the test will be uploaded to Lev El's server (might take a few minutes).
7.3 Wait for the server to analyze the test.

8. **Third tab (Generate Report)** – press **Load Results** to see your tests statuses:
- New Patient – new patient enrolled.
 - In process – waiting for the server to analyze.
 - Ready to generate PDF – test result is ready.
 - Finished – test result was opened.

9. **Third tab (Generate Report)** – **Open** the relevant test and press **Generate PDF** to create and save the result report:



**This Quick Guide does not replace the full user guide. Please review Lev El Client User Guide

**The full user guide and a video tutorial is available at Lev El's website.